

**IMPORTANT- PLEASE READ BEFORE USE:**

- NO FOOD OR DRINK WHEN TESTING. Remove all snacks and water bottles from testing area.
- Please check lot number and expiration dates before use **DO NOT** use if expired
- **DO NOT** perform Ag testing if patient using Mupirocin (Bactroban & Centany)
  - o It is recommended to perform SARS-CoV-2 PCR testing on these patients
- **CAUTION:** Treat all collected samples as infectious
- **CAUTION:** Carefully insert swab and spin to avoid reagent splash
- **DO NOT** interchange components between kits
- **ONLY USE SWABS that come with the kit**
- **DO NOT remove test card from foil until you are ready to do the test**
- Immediately perform test after collection
- **DO NOT** store swabs in original paper packaging (could result in false test results)
- **READ RESULTS** promptly between 15-minutes and 30 minutes
  - o **DO NOT Read Results before 15 minutes or after 30 minutes when performing test**
- **DO NOT** interpret results if no control line is visible
- **ALL RESULTS ARE REPORTABLE TO MDSS**

**Items to have before you begin testing:**

1. PPE
  - a. N95 respirators (fit testing required **PLEASE CONSULT CDC PPE GUIDELINES**) or other respiratory protection
  - b. Gloves
  - c. Eye protection: face shield or goggles
  - d. Disposable coat recommended
2. Rapid COVID-19 Antigen Testing Results BinaxNOW Reporting Instruction sheet
3. Laptop
4. Disinfectant with paper towel/wipes
5. Hand sanitizer
6. Pens
7. Timer or wall clock. Do not use personal cell phone as a timer, to avoid contamination with COVID-19.
8. Permanent markers
9. Biohazard Waste container (**follow local and MIOSHA regulations**)